GPARMART

MEDICAL COMPRESSION STOCKINGS


Open Toe Knee High

Closed Toe Knee High


Open Toe Thigh High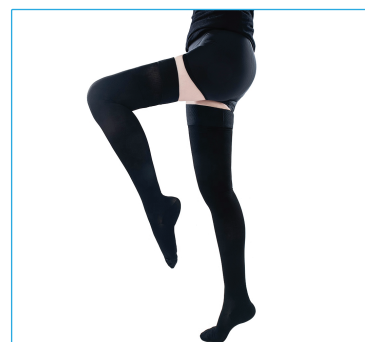


Closed Toe Thigh High

Footless Thigh High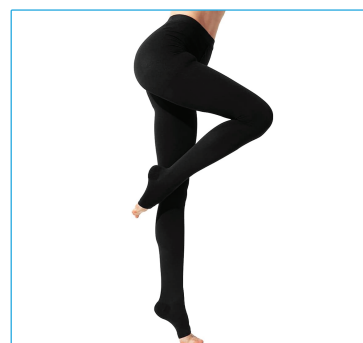

Open Toe Pantyhose

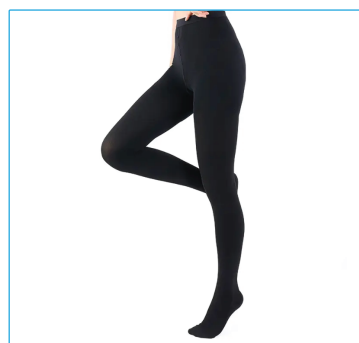Closed Pantyhose

Footless Pantyhose

SCAN ME!

(Please Find Online User Instruction Manual)

GPARMART

- Thank you for purchasing GPARMART Compression Stockings. Please read the user guide before using stockings.

PRODUCT RANGE

- ➔ Knee-High Compression Stockings - Open / Closed Toe
- ➔ Thigh High Compression Stockings – Open / Closed Toe / Footless
- ➔ Pantyhose Compression Stockings - Open / Closed Toe / Footless

COMPRESSION LEVEL

- 23–32 mmHg, Class 2 Medical Compression Stocking

SIZES AVAILABLE (IN 6 SIZES)

- S, M, L, XL, XXL, 3XL (Measure according to size chart shown on Amazon)

COLORS AVAILABLE

- BLACK, BEIGE

PRODUCT USAGE

- To improve blood flow & lower the chances of getting deep vein thrombosis (DVT) Moderate varicose veins, or significant swelling, fatigued legs.

MATERIAL SPECIFICATIONS

- Nylon 70%, Spandex 30%

STORAGE CONDITIONS

- Store in dry place, room temperature

HOW TO WEAR

- ➔ Closed Toe Stockings:
 - Hold the top of the stocking and roll it down to the heel.
 - Put your foot into the stocking as far as you can
 - Pull the stocking up
- ➔ Open Toe Stockings – Use aid to pull the stocking on thighs

TAKING CARE OF STOCKINGS

- ➔ Do not touch socks with sharp objects.
- ➔ Wash regularly, air dry, replace regularly every 3–6 months, since elasticity wears out over time.

HOW TO USE STOCKINGS FOR BEST RESULT

- ➔ Start by wearing compression stockings for a few hours each day and increase the number as you feel more comfortable
- ➔ Take off stockings before going to bed
- ➔ Wear stockings when you wake up in the morning

GPSR: RESPONSIBLE PERSON CONTACT DETAILS


- GPARMART AB
- Stupvägen 49
191 42 Sollentuna
- SWEDEN
- Email: info@gpalmart.com

COUNTRY OF ORIGIN

- China

INSTRUCTIONS FOR SAFE DISPOSAL

- Regular disposal of household waste is possible

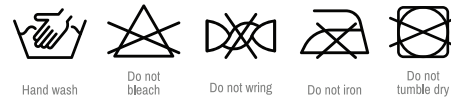WASHING INSTRUCTIONS**PACKAGE CONTENT**

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